



VITRIFICATION BY ORIGIO®

DMSO Free Vitrification Media

For Efficient Embryo Cryopreservation

- Simple and easy-to-follow protocol
- Ready to use media
- DMSO free



origio

a CooperSurgical Fertility Company

Medicult Vitrification Media For Efficient Embryo Cryopreservation

Vitrification

Vitrification is a cryopreservation technique based on ultrafast cooling. High cryo-protectant concentrations in the media rapidly dehydrates the cells, before they are plunged into liquid nitrogen. This rapid cooling solidifies the cell into a "glass-like state" without any ice formation.

MediCult Vitrification Cooling and Warming

The media has been designed for vitrification, storage and warming of human embryos. The media contains 12mg/ml HSA and the cryoprotectants PROH, EG and Sucrose.

Benefits of MediCult Vitrification Media

- Simple and easy-to-follow protocol
- Ready to use media
- DMSO free

Catalog No.

1228 4001	MediCult Vitrification Cooling, 4x1ml
1229 5002	MediCult Vitrification Warming, 5x2ml

Note: The products are provided in vials intended for single use.

Recommended Vitrification Carriers

VitriGuard (pack of 20)

4378 2001	VitriGuard Clear
4379 2001	VitriGuard Blue
4380 2001	VitriGuard Green
4381 2001	VitriGuard Yellow

Clinical data with DMSO free media

	Number	Survival rate (%)	Reference
Embryos	123/29	91.1 / 89.7	Dundure et al., 2010, Hum. Reprod.
	97	90	Phillips et al., 2010, ALPHA, Budapest
	83	85.5	Son et al., 2009, Fert. Steril.
	40	97.5	Fernandez et al., 2007

Optimize results by:

- Vitrifying good material only
- Mixing vials well before use
- Maximum 60 seconds exposure to vitrification medium
- Loading cells onto open device in <1µl volume
- Keeping vitrified cells under LN₂ at all times

Quality control testing

- Sterility tested
- Endotoxin tested ≤ 0.5 EU/ml
- Mouse Embryo Assay (MEA) tested
- pH tested
- Osmolality

Note: The results from each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

Contact

For further information on products or the possibility of arranging a hands-on vitrification workshop, please visit www.origio.com to find your local ORIGIO distributor or contact ORIGIO directly at the address listed below.

Learn more about culture at www.origio.com



Tel: 1-800-648-1151
www.origio.com

CooperSurgical Fertility Companies